Tensoval[®]

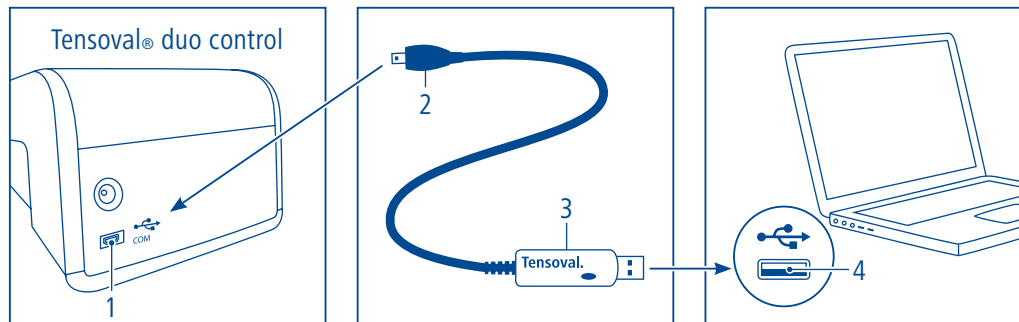
cardio control online

Инструкция по
эксплуатации
Návod k použití
Návod na použitie
Instrukcja obsługi
Manual de utilizare
Használati utasítás
Kullanım kılavuzu
Instructions for use

HARTMANN

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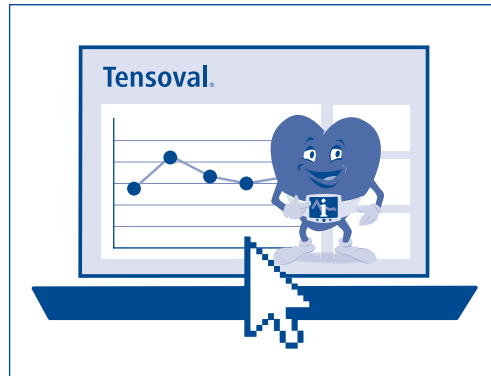
Dear customer,

Thank you for deciding to purchase Tensoval® cardio control online from HARTMANN. Tensoval® cardio control online allows you to transfer your personal blood pressure measurement values from our Tensoval® duo control blood pressure monitor to the Tensoval® Online-Portal.

After registration on the portal, you can transfer, save, compare and analyse your measurement values with the aid of the cable in a simple and convenient manner.

We hope you enjoy using Tensoval® cardio control online!

The Tensoval® Online-Portal provides a quick overview of the history of your measured values.

 **Please note:** In addition to the measured blood pressure values, the date and time of the measurement are also transferred.

Please check these settings prior to measuring on the display of your blood pressure monitor to ensure that the values are saved with the correct date and the correct time. If necessary, the date and time can be set easily.

1. Prerequisites for use

Computer with at least one free USB-1.1/2.0 type A port.

■ Supported web browsers:

Google Chrome 27, Mozilla Firefox 21, Apple Safari 6.1, Microsoft Internet Explorer 7 and newer versions of these web browsers

■ Supported operating systems:

Windows XP or later, MAC OS X 10.5 or later, Linux

Note: As a rule the data transfer does not function with Smartphones and tablet computers.

2. Installation

Prior installation of a driver software is not necessary.

Step 1: Connecting the cable

■ Connect the data transfer cable to the computer and the Tensoval® duo control blood pressure monitor. The device will then be recognised by your operating system.

Step 2: Calling the Tensoval® Online-Portal

■ Microsoft Windows: Depending on the version and settings, the "Tensoval" drive will open automatically. If this does not happen, open My Computer and double click on the drive named "Tensoval".

Double click on the file "START". A browser window will open and the homepage will appear.

■ Other operating systems: Open your web browser and enter www.tensoval.com/app.

Step 3: Registration (during first visit)

■ Click on "Register" and enter the required data. Then follow the instructions.

Step 4: Using the Tensoval® Online-Portal

■ You can upload your measured values via the "Upload" button on the Tensoval® Online-Portal. To start the upload, you must select the "Tensoval" drive and the file "Tensoval.csv" on this drive. The upload is followed by an analysis of the measured

values. A supporting help function makes the Tensoval® Online-Portal especially user-friendly.

Step 5: Exiting the Tensoval® Online-Portal

■ Click on "Logout" to exit from the online portal.

3. Important notes

- To transfer data, the data transfer cable must be connected to both the computer and the blood pressure monitor.
- The data is transferred in encoded form and not saved on the data transfer cable.
- The data transfer cable may not be used in connection with other devices!

4. Warranty conditions

We give a 2-year warranty on this high-quality data transfer cable from the day of purchase and in accordance with the following conditions. Claims must be made during the warranty period. The date of purchase must be documented by proof of purchase. Within the warranty period, HARTMANN shall replace or repair any faulty

device components free of charge which were caused by material or manufacturing errors. This does not extend the warranty period. The warranty does not cover damage resulting from improper use or unauthorised treatment. Compensation for subsequent damage is expressly excluded. In the case of a warranty claim, please send the data transfer cable directly or through your specialist retailer to the responsible customer service in your country.

(You will find the contact address in chapter 10.)

5. Meaning of the symbols on the packaging

 Pay attention to the operating instructions

 Please note

 Manufacturer

 Electronic device disposal note

 Disposal note

 CE marking

6. Technical data

Energy supply:	via USB; no external power supply required
Operating voltage:	DC 5 V ± 5 % (by the USB port of the connected computer)
USB port:	USB version 2.0, compatible with USB version 1.1
Protection against harmful ingress of foreign bodies and water:	IP20 (not protected from moisture)
Storage / transport conditions:	-20 °C to 50 °C, 15 % to 90 % rel. humidity, non-condensing
Operating conditions:	10 °C to 40 °C, 15 % to 90 % rel. humidity, non-condensing
Weight:	40 g
Cable length:	approx. 1.5 m
Serial number:	see product label

7. Instructions for disposal

This product is subject to European Directive 2002/96/EC or 2012/19/EU on used electrical and electronic equipment and is marked accordingly.

Never dispose of electronic equipment together with your household waste. Please obtain information concerning the local regulations on the proper disposal of electrical and electronic products. Proper disposal protects the environment and human health.

8. Safety instructions

- Ensure that the USB cable is only connected to a computer. Otherwise the functioning is not guaranteed.
- Never open or take apart the housing of the cable.
- Never expose the USB cable to moisture or liquids.
- It is essential that you prevent any damage to the cable.
- You may neither modify nor repair the cable yourself.

9. Legal requirements and guidelines

Tensoval® cardio control online bears the CE marking and complies with European Directive 2011/65/EU on the restriction of the use of certain hazardous materials in electrical and electronic equipment.

10. Contact information for customer queries

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AU – PAUL HARTMANN Pty. Ltd.
GB – PAUL HARTMANN Ltd.
Asia-Pacific Ltd.
PAUL HARTMANN AG · 89522 Heidenheim, Germany
TR – PAUL HARTMANN Ltd. Sti. · 34746 Ataşehir, Istanbul